

An Innovative Rapid Test Designed by WANTAI



Rapid Test for Antibody to Syphilis (Colloidal Gold Device)



WANTAI

Leading diagnostic test
manufacturer of infectious
diseases

Your Global Partner in IVD



***Developing Scientifically
Focusing on the Health***



Innovative



High
Performance



Easy-to-use




Rapid

Test Process


1. Prepare Test

Allow the test cassette to reach room temperature, use it within 20 minutes after opening the pouch.

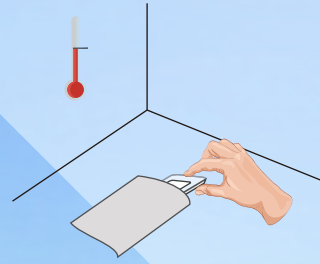


2. Add Specimen

1) **For whole blood specimens**
Add one drop of blood
Add 40-80μl of Diluent Buffer

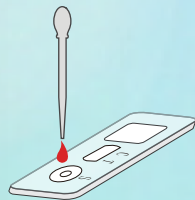


40μl of specimen

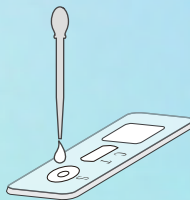

40-80μl of diluent buffer


2) **For the serum/ plasma specimens**
Add two drops of blood

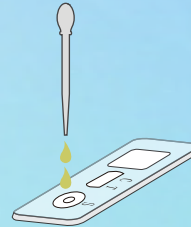

80μl specimens of S/P

3. Read Result


Read the results within 30 minutes.

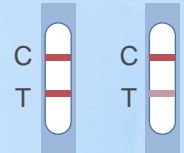

Positive



Negative

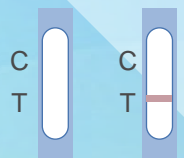

Invalid

* Please refer to product insert for detailed instruction.

Advanced Diagnostic Technology

- 01 Lateral Flow Technology:** makes it possible to perform rapid on-site testing
- 02 Superior Performance:** 99.63% sensitivity and 99.93% specificity
- 03 Easy-to-use:** No special equipment required
- 04 Fast:** clear answer within 30 minutes only
- 05 All in one:** all accessories in the kit allows to do point-of-care testing
- 06 Room Temperature Storage:** 18 months shelf-life

Order Information

Components	WJ-1610	WJ-1650	WJ-1610E	WJ-1650E
Test Cassette	x 10	x 50	x 10	x 50
Diluent Buffer	x 1 vial	x 2 vials	x 1 vial	x 2 vials
Safety Lancet			x 10	x 50
Disposable Pipette			x 10	x 50
Alcohol Pad			x 10	x 50