

An Innovative Rapid Test Designed by WANTAI

SELF TEST for


Antibody to HIV
Human Immunodeficiency Virus (Urine)



WANTAI

Leading diagnostic test
manufacturer of infectious
diseases

Your Global Partner in IVD

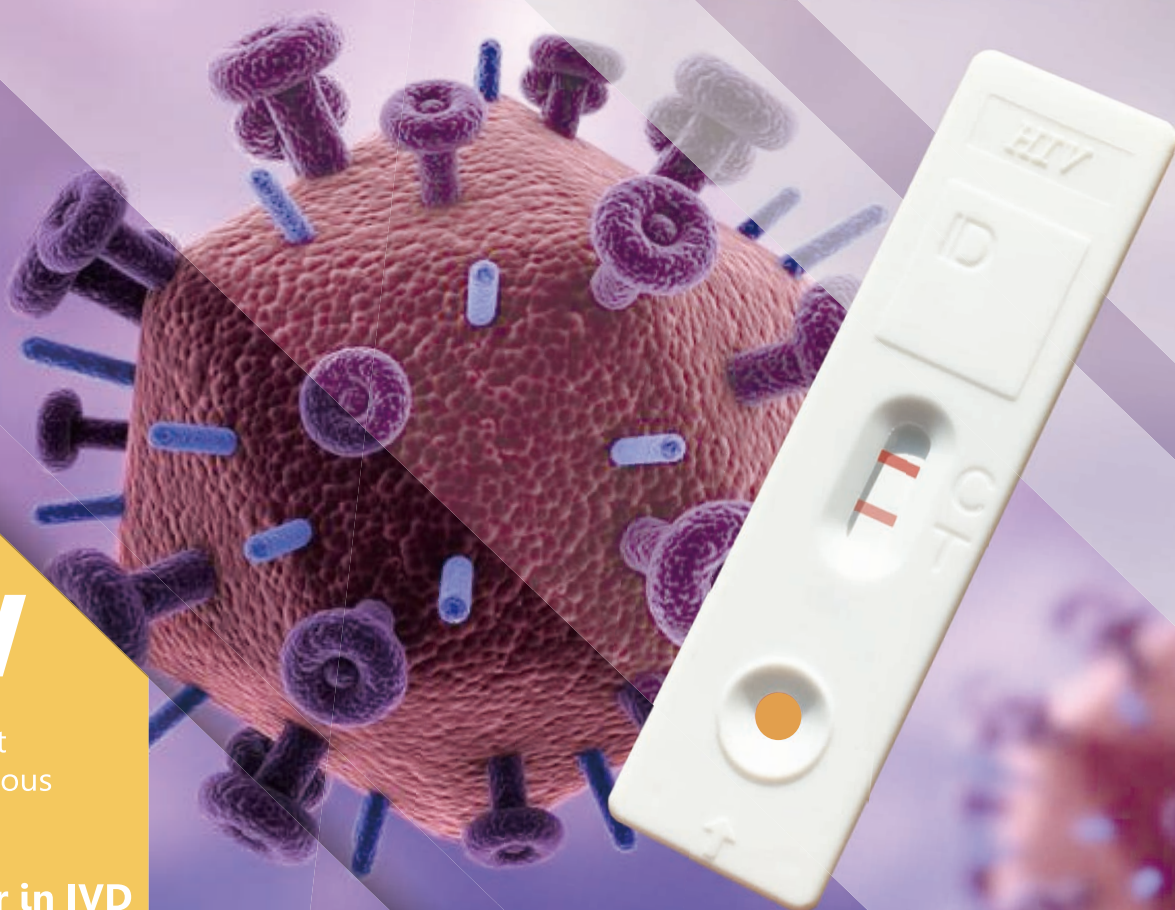

*Developing Scientifically
Focusing on the Health*



Innovative



High
Performance



Easy-to-use



Rapid

Test Process

① Collect urine

in the cap of the test kit package.



② Open

the pouch of the test cassette. Place the cassette on a flat surface.




③ Absorb sufficient

amount of urine with the provided plastic pipette.




④ Add **three (3)** drops of

urine into the cassette sampling zone. The sampling zone is marked with a small arrow. Do not overflow.

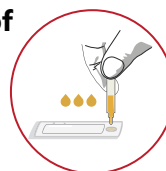

⑤ Wait


15 minutes. Do not move the test cassette. After 15 minutes you can read the results.

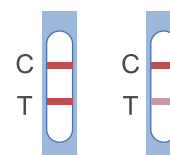

Positive



Negative

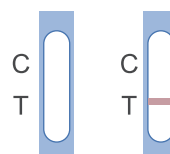

Invalid

* Please refer to product insert for detailed instruction.

Advanced Diagnostic Technology

- ① **Lateral Flow Technology:** makes it possible to perform rapid one-site screening
- ② **Superior Performance:** accuracy > 99%
- ③ **Easy-to-use:** No special equipment. Works with urine samples
- ④ **Fast:** clear answer in 15 minutes
- ⑤ **All in one:** all accessories in the kit allows to do point-of-care testing
- ⑥ **Room Temperature Storage:** 18 months shelf-life

Order Information

Components	WZ-1001
Test Cassette	x 1
Disposable Pipette	x 1