

An Innovative Rapid Test Designed by WANTAI


Rapid Test for Total Antibody to SARS-CoV-2 (COVID-19)



WANTAI

Leading infectious diseases
diagnostic test manufacturer

Your Global Partner in IVD

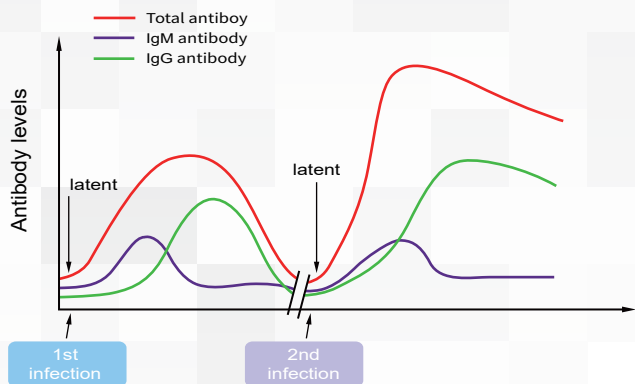

Antibody dynamics during COVID-19 infection



Wantai SARS-CoV-2 total antibody rapid test is intended for **screening of patients** suspected for infection with the SARS-CoV-2 virus, an aid in the diagnosis of the **COVID-19 disease**.



Innovative



High Performance



Easy-to-use




Rapid

Test Procedure



1. Prepare the Test

Allow the test cassette to reach room temperature, use it within 20 minutes after opening the pouch.

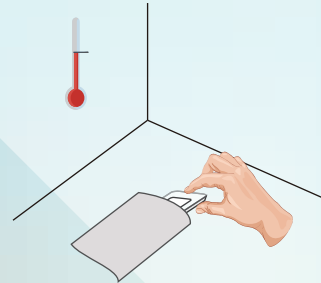


2. Add the Specimen

Add 10µl of whole blood, serum or plasma specimen
Add two drops of Diluent Buffer.

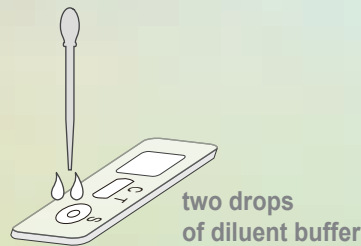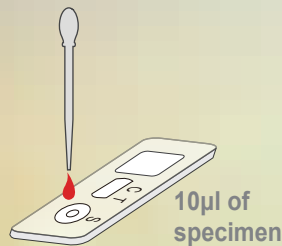

3. Read Result


Read the results at 15 min.

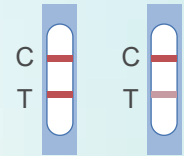

Positive



Negative

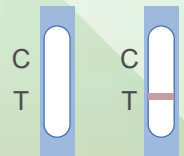

Invalid



* Please refer to product insert for detailed instruction.

Advanced Diagnostic Technology

01 Lateral Flow Test - suitable for rapid, on-site testing

02 Performance: sensitivity of 94.70% (125/132) and specificity of 98.89% (268/271). The test has been clinically validated during the 2020 COVID-19 outbreak in China

03 Easy-to-use: No special equipment. Works with whole blood, serum or plasma samples

04 Fast: clear answer within only 15 minutes

Order Information

Components	WJ-2710	WJ-2750
Cassette	x 10	x 50
Diluent	x 1	x 5