

An Innovative Rapid Test Designed by WANTAI



Rapid Test for IgM Antibody to Hepatitis A Virus



WANTAI

Leading diagnostic test
manufacturer of infectious
diseases

Your Global Partner in IVD

*Developing Scientifically
Focusing on the Health*



Innovative



High Performance



Easy-to-use




Rapid

Test Process


1. Prepare Test

Allow the test cassette to reach room temperature, use it within 30 minutes after opening the pouch.

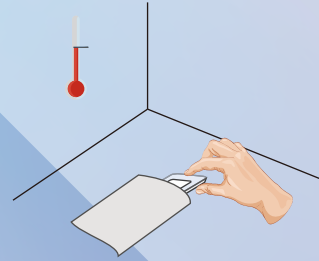


2. Add Specimen

1) Pipette 5 μ l of serum or plasma into the Sample Dilution Tube and mix sufficiently.

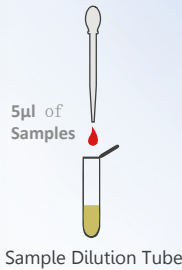


2) Pipette 80 μ l of diluted specimen into the sample well on the cassette.

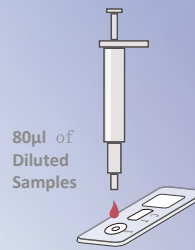

3. Read Result


Read the results in 10 minutes.

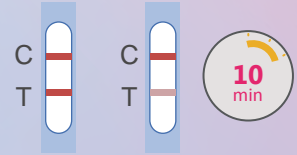

Positive



Negative

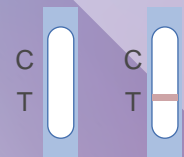

Invalid

* Please refer to product insert for detailed instructions.

Advanced Diagnostic Technology

- 01 **Lateral Flow Technology:** makes it possible to perform rapid on-site testing
- 02 **Superior Performance:** 100% sensitivity and 97% specificity
- 03 **Easy-to-use:** No special equipment. Works with serum and plasma samples
- 04 **Fast:** clear answer within 10 minutes only
- 05 **All in one:** all accessories in the kit allows the user to do point-of-care testing
- 06 **Room Temperature Storage:** 18 month shelf-life

Order Information

Components	WJ-1010	WJ-1050
Cassette	x 10	x 50
Diluent	x 10	x 50